# Breast Pump Kit Care

## Cleaning – Drying – Storing

Good hygiene will help protect the breast milk that you give to your baby. For the best protection when handling your breast milk, wash your hands with soap and water, and follow the guidelines below for cleaning, drying and storing your kit.

*If your baby is premature or ill, consult with your healthcare provider for further cleaning and storage instructions.*

*See Medela's instruction booklet for detailed kit assembly, cleaning & pumping information.*

### How do I clean my breast pump kit?

#### 1. Finish pumping.

- While the pump is still running, disconnect tubing from breast shields.

- Continue to let pump run for 2 minutes with tubing attached. This will eliminate condensation that naturally results from humidity.

#### 2. Take apart kit and store breast milk.


- With tubing still connected to pump, separate all parts of your kit that come in contact with your breast or breast milk.

**A.** Remove bottles from breast shields and store breast milk.

**B.** Twist and pull the yellow valves off the breast shields.

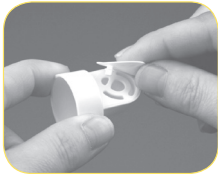**C.** Remove the white membranes from the valves.

**D.** Separate breast shields if using the PersonalFit™ breast shields that have two pieces.

#### 3. Wash Kit.

- Rinse all separated parts in cool water to remove breast milk
- **Wash and Rinse** -or- **Dishwasher**
  - Wash all separated parts in warm, soapy water.
  - Wash all separated parts on top rack of dishwasher.
  - Rinse all separated parts with clear water.

If sanitizing your kit, separate all parts that come in contact with your breast or breast milk and boil for 10 minutes or follow the instructions on Medela's Quick Clean™ Micro-Steam™ bag.

### Quicker Cleaning

You can use Medela's Quick Clean wipes in between pumping sessions and then clean your kit once you get home. Follow instructions on Quick Clean package.

### Drying & Storing

Allow all breast pump kit parts to air dry in a clean area and store dry parts when not in use. Do **NOT** store wet or damp parts.

### When do I care for my tubing?

If your tubing comes in contact with breast milk clean it by following these directions.

- Rinse and run water through tubing to remove breast milk.
- Dry tubing by attaching it to pump and running for 2 minutes or until dry. This will eliminate condensation that naturally results from humidity.

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